

Schweizer

Spezialbetten · Spezialsessel e.K.

Wolfram Schweizer
Dörrersgasse 12
90403 Nürnberg
Telefon 0049-911-22 52 18
Fax 0049-911-22 28 27
info@wolframschweizer.de
www.spezialbett.com



RotoFlex®
will launch a new quality
in hospital beds



RotoFlex®
health care has a name



Schweizer

Spezialbetten · Spezialsessel e.K.

Wolfram Schweizer
 Dörrersgasse 12
 90403 Nürnberg
 Telefon 0049-911-22 52 18
 Fax 0049-911-22 28 27
 info@wolframschweizer.de
 www.spezialbett.com



The RotoFlex® bed is a high end bed designed to assist a patient and enhance quality of life, comfort, and freedom of movement, by enabling a patient to leave the bed without assistance from a nurse. On the other hand, the bed permits easy and safe nursing assistance depending on the individual disability.

The performance of the RotoFlex® bed addresses the health care of the patient on the one hand and the costs for nursing personnel on the other. In hospitals, the RotoFlex® bed is useful in neurology and geriatric departments, post abdominal surgery, and after child birth, to mention only a few areas.





Wolfram Schweizer
 Dörrersgasse 12
 90403 Nürnberg
 Telefon 0049-911-22 52 18
 Fax 0049-911-22 28 27
 info@wolframschweizer.de
 www.spezialbett.com

All RotoFlex® beds have in common allowing a rotation of the mattress support about a vertical axis. At the end of the rotation, the mattress is oriented transverse to the bed. In this position the mattress can be folded into a chair-like conformation. Now the RotoFlex® bed provides a seating surface and a leg rest to the patient or user.

In case of RotoFlex® beds type RO an electric motor is used to achieve the rotation of the mattress support. All movements of the electric motors involved while bringing the bed from the bed position into the chair conformation and vice versa are computer controlled.

With the RotoFlex® beds type RC, the rotation is carried out manually by a care giver.

With all types of RotoFlex® beds, the movements of the back rest, the leg rest and the base for raising and lowering the bed are driven by electric motors. At the manufacturers end, all types of beds can be equipped with a tube system for a particular adapted anti-decubitus air mattress.

RotoFlex® RO-H

RotoFlex® RO-H is designed for use in hospitals. The bed provides all essential features necessary in hospitals. The head board can be detached. The bed is provided with large diameter castors including a central lock and directional lock. Moreover, RotoFlex® RO-H includes a virtual hinge for the back rest.

The lower base frame of the bed is designed for heavy duty application where the bed is generally used for transportation of a patient.

Accessories rotating along with the mattress support like i.v.-pole, urine bag holder, side rails, bed extension and cushion support are available. The bed can be equipped with a CPR-function of the back rest at the manufacturer's end.

RotoFlex® RO-P

RotoFlex® RO-P bed shows a more furniture-like appearance. It includes wooden sideboards. The head and the footboard are also made of wood. All wooden parts are attached to the upper frame of the base and will move up and down along with the mattress. RotoFlex® RO-P can be equipped with castors of different diameters with or without central lock.

**HEADBOARD
REMOVABLE**

Wolfram Schweizer
Dörrersgasse 12
90403 Nürnberg
Telefon 0049-911-22 52 18
Fax 0049-911-22 28 27
info@wolframschweizer.de
www.spezialbett.com



Calculation example: RO-1003-H HOSPITAL

RotoFlex Sit and Rise bed RO-1003H on castors 125 mm with central brake system, wooden/steel surround RONDO STEEL/dekor beech	12.500,00
1 visco-elastic mattress in an incontinence cover with 3-sided zip fastener, with anchor system for RotoFlex Sit and Rise bed	850,00
2 grab handles can be swivelled for use on left or right	374,00
Pull handle with height adjustable trapeze bar (2-parted)	150,00
Lathflor 90 X 210 cm	
Ex works Germany in EURO	13.874,00
Packaging in a wooden box for sea or airfreight, ready mounted for use	300,00

Wolfram Schweizer
 Dörrersgasse 12
 90403 Nürnberg
 Telefon 0049-911-22 52 18
 Fax 0049-911-22 28 27
 info@wolframschweizer.de
 www.spezialbett.com

**HEADBOARD
REMOVABLE**



Calculation example: RO-1003-H HOSPITAL

-RotoFlex Sit and Rise bed RO-1003H on castors 125 mm with central brake system, Wooden/steell surround RONDO STEEL / décor beech Lathflor 90 X 210 cm	10.500,00
-1 visco-elastic mattress in an incontinence cover with 3-sided zip fastener, with anchor system for RotoFlex Sit and Rise bed, 11cm high	850,00
-2 grab handles can be swivelled for use on left or right	374,00
-2 side rails for back position	340,00
-1 fitted sheet, towelling, jersey	82,00
-1 Pull handle with height adjustable trapeze bar (2-parted)	150,00
-1 Infusion pole with adapter	436,00
-1 urin bag holder	83,00
Ex works Germany in EURO	14.815,00
Packaging in a wooden box for sea or airfreight, ready mounted for use	300,00

Schweizer

Spezialbetten · Spezialsessel e.K.

Wolfram Schweizer


Dörrersgasse 12
90403 Nürnberg

Telefon 0049-911-22 52 18

Fax 0049-911-22 28 27

info@wolframschweizer.de

www.spezialbett.com

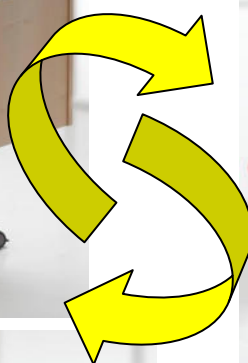

Calculation example: RO-1003-PSB RONDO

RotoFlex Sit and Rise bed RO-1003P on castors 75 mm with cover and wooden surround RONDO type III in maple, additional charge	12.335,00
Mattress Trioptima visco-elastic mattress in an incontinence cover with 3-sided zip fastener, with anchor system for RotoFlex Sit and Rise bed	1.100,00
2 grab handles can be swivelled for use on left or right	986,00
Lathflor 90 X 210 cm	374,00
Ex works Germany in EURO	14.795,00

Packaging in a wooden box for sea or airfreight, ready mounted for use 300,00

Schweizer

Spezialbetten · Spezialsessel e.K.

Wolfram Schweizer
 Dörrersgasse 12
 90403 Nürnberg
 Telefon 0049-911-22 52 18
 Fax 0049-911-22 28 27
 info@wolframschweizer.de
 www.spezialbett.com

Individually designs



For heavy people:

RotoFlex as heavy duty bed up to 300 Kg
 in width 100 or 120 cm X 210 cm

- additional charge EURO 3.500,00
- All accessories + 25%

accessories are broadened and boosted
 (Not for steel reling)



Calculation example: RO-1003-PSB DIVAN

RotoFlex Sit and Rise bed RO-1003P on castors 75 mm with cover and wooden surround DIVAN in walnut / white, additional charge	12.335,00
fastener, with anchor system for RotoFlex Sit and Rise bed	1.100,00
2 grab handles can be swivelled for use on left or right	374,00
Headrest lifting	280,00
Cold foam-mattress, HD-foam airpermeable, normal strength, for bodyweight 50-90kg	640,00
Lathflor 90 X 210 cm	
Ex works Germany in EURO	14.729,00

Optional:

Commode with castors, door and top drawer in walnut	1.150,00
---	----------

Packaging in a wooden box for sea or airfreight, ready mounted for use 300,00

Schweizer

Spezialbetten · Spezialsessel e.K.



Wolfram Schweizer
Dörrersgasse 12
90403 Nürnberg
Telefon 0049-911-22 52 18
Fax 0049-911-22 28 27
info@wolframschweizer.de
www.spezialbett.com

Individually designs — **Homely** wooden bedframe with cutout, Rondo II in walnut dark RO-Z2024 PII with hidden castors in 75mm and brakes



SLEEPING



RELAXING



**GETING UP
50 - 90 CM**



SITING

Calculation example: RO-1003 **Homely** wooden bedframe with cutout, Rondo II

RotoFlex Sit and Rise bed RO-1003P on castors 75 mm with cover and wooden surround DIVAN in walnut dark, additional charge	12.335,00
fastener, with anchor system for RotoFlex Sit and Rise bed	1.500,00
2 grab handles can be swivelled for use on left or right	374,00
Headrest lifting	280,00
2- parted mattress with 3-D tissue and visco-foam cover for highest sleeping quality, Height 14 cm	2.000,00
Lathflor 100 X 210 cm	270,00
Lifting pole with handle standart	150,00
Ex works Germany in EURO	16.629,00

Packaging in a wooden box for sea or airfreight, ready mounted for use 300,00

**RotoFlex as heavy duty bed up to 300 Kg
in width 100 or 120 cm X 210 cm**

- additional charge EURO 3.500,00
- All accessories which must be widened
+ 25% (mattress and not serial frames)

accessories are broadened and boosted
(Not for steel reling)

Wolfram Schweizer
Dörrersgasse 12
90403 Nürnberg
Telefon 0049-911-22 52 18
Fax 0049-911-22 28 27
info@wolframschweizer.de
www.spezialbett.com

Bariatric bed up to 300 Kg personal weight, lathflor 100 cm or 120 cm X 210 cm



Calculation example: RO-1003 PSP

RotoFlex Sit and Rise bed RO-1003P Rondo I in décor beech	12.335,00
Central brake with 100 mm castors	750,00
2 grab handles can be swivelled for use on left or right	374,00
Headrest lifting	280,00
Lifting pole with handle standart	150,00
mattress with viscoelastic-foam, height 14cm, bodyweight more than 100 Kg	1.200,00
Heavy duty additional charge for 100 cm width	3.500,00
Accessories additional charge (mattress) <u>with 120 cm width</u>	300,00
Ex works Germany in EURO	18.889,00
Packaging in a wooden box for sea or airfreight, ready mounted for use	300,00

Wolfram Schweizer
Dörrersgasse 12
90403 Nürnberg
Telefon 0049-911-22 52 18
Fax 0049-911-22 28 27
info@wolframschweizer.de
www.spezialbett.com

Specifications of the sit and rise bed RotoFlex

Overall dimensions: length 2210 mm / width 950 mm

Bedding area: 2100 X 900 mm

Lathfloor: each section with compact plate top,

Optional 32 glasfiber springbars

Mattress frame: the mattress frame ist sectionated in six

Part – head rest section, back rest section

Sitting section, upper leg section, lower leg

Section and feet section.

Backrest length: 850 mm

Backrest adjustment: 0-85° if headrest locked, up to 92°

Emergency reset of the backrest: either by electrical drive or manually

Sitting length: 190 mm

Thigh rest length: 275 mm

Thigh rest adjustment: 0-50°

Lower leg rest length: 375 mm

Lower leg rest adjustment: -95° - 0°

Footrest: not adjustable

Height of the upper side of mattress in lowest position: ~ 500 mm

(depending of wheels and mattress) above floor

Height of the upper side of mattress in lifted position: ~ 900 mm

(depending of wheels and mattress) above floor

Chair position: with slightly evelated upper legs

Rotation direction of the mattress frame in from the sitting position:

Mattress frame rotatable 90° clockwise or anticlockwise. The rotation

Direction of the mattress frame is preset. Changing of the direction

Requires trained maintenance personal

Safe workload: 170 Kg incl. mattress

Total weight of the bed: ~ 200 kg, depending of options

Undercarriage clearance: 120 mm (depending of wheels(castors))

Castors/wheels: 75mm, 100mm and 125 mm optional

Mattress: sectionated in two parts, main part moving with the associated

Part of the mattress frame, foot part remains stationary in bed.

Mattress fixation: the mattress is fixed to the lath floor by a anchor system

Two layer foam: basis is an 4 cm foam type T4072

Upper part ist 7 or 10 cm open cell foam with heavy

Densitiy type 5542

Opitonal upper layer with viscoelastic foam

Type GV80080

Mattress measurements: Body section 1730 mm X 880 mm

Foot section 350 mm X 880 mm

Thickness 110 mm or 140 mm

Wolfram Schweizer
 Dörrersgasse 12
 90403 Nürnberg
 Telefon 0049-911-22 52 18
 Fax 0049-911-22 28 27
 info@wolframschweizer.de
 www.spezialbett.com

Get out of bed with the ROTOFLEX® – a step back into independence

The ROTOFLEX® is an innovative nursing bed which was developed to improve the quality of life, independence, comfort and mobility of patients. It allows those affected to get out of bed without help from others – so the independence is maintained and having to leave the familiar surroundings is avoided.

The familiar private surroundings of the patient are maintained.

Relief of the nursing staff – more time for the patients

To mobilise those affected and to implement other nursing measures is with the ROTOFLEX® not only easier, but also gentler – both for the user and for the nursing staff: transfer and mobilisation of the patient with the stand-up bed is without stress and easy on the back. Time constraints and physical strain on the nursing staff are significantly reduced. Many measures can be done with less time and personnel costs, thus providing additional time for patient-related activity.

The ROTOFLEX® saves costs and at the same time provides more time to meet the individual needs of patients.

Getting out of bed – a barrier is overcome

Getting out of bed means for people with a movement impairment the overcoming of an invisible barrier. Up till now, they were only able to overcome this with the help of others. The ROTOFLEX® overcomes the hurdle, which forces the affected person to lie down in bed and obstructs them from access to activities outside the bed.

Four strong, reliable motors lift them up at the touch of a button, program-controlled, softly and quietly for sitting on the edge of the bed and then back to lying down. All movements of the ROTOFLEX® can also be individually and comfortably controlled via the remote control so that the five-part reclining surface can be steplessly brought to any desired position.

Nursing staff are relieved considerably by the ROTOFLEX®, among other things, with the patient transfer into a wheelchair or a standing aid, body-distant lifting is omitted, down-times are minimised and a longer stay in the occupation becomes possible.

The ROTOFLEX® paves the way back to independence.



Wolfram Schweizer

Dörrersgasse 12
90403 Nürnberg

Telefon 0049-911-22 52 18

Fax 0049-911-22 28 27

info@wolframschweizer.de

www.spezialbett.com

Specifications of the sit and rise bed RotoFlex

Motors: Dewert 24V CD IPX 44

Handcontrol: Dewert

Gasspring: Hahn

Castors: Steinco

Glasfibersprings: Fibrolux, flame resistant

Upper cover frame: PCV grey, flame resistant

Steel parts: Carbon steel powder

Colour: light grey, RAL 7035

Brake system: central controlled blocking system for wheels of all the castors,
Operated by 2 pedals at the foot or optional single brake castors

Sterilisation of the bed: manually, no washing machines

Power supply: 230V / 1,6 Amper, Optional: 110Volt / 32,2 Amper

Protection: IP44, class II

Output voltage: 24V

Power limit: 5,6 A

Power on time of the motors: max. 2 minutes / 18 min (automatically safety stop)

Power consumption: max. 200 VA

Adjustment speed for the motors: max. 5,4mm / sec.

Primary fusing: automatic

Secondary fusing: automatic

Thermal fusing: yes

Thermal switch against overheating: yes

Compliance list, according to:

EU guideline 93/42/EWG

CE mark for this guideline: yes

Electrical safety: DIN EN 60601-1

Electromagnetic compatibility EMC: DIN EN 60601-1-2

Safety of electrically operated: DIN EN 60601-2-38

Safety of the beds for handicapped persons: EN 1970-2000

Compliance with german medical product act

Back part up
and down

Inner footboard up
and down

Stand up aid
up and down

Rotation in or out
of the bed



Delivery time **in stock: 5-8 days**
Air freight ~ 10 days
Sea freight ~ 5 weeks

Delivery time **personalized: 6 weeks EX works**
Air freight ~ 10 days
Sea freight ~ 5 weeks

Schweizer

Spezialbetten · Spezialsessel e.K.



Wolfram Schweizer
Dörrergasse 12
90403 Nürnberg
Telefon 0049-911-22 52 18
Fax 0049-911-22 28 27
info@wolframschweizer.de
www.spezialbett.com

OPTIONS

Infusion pole
RO-2034

Urin bag holder
RO-Z2011

Height-adjustable footrest
RO-Z2117

Pillow pocket
RO-Z2014K




Turning with the bed



Headrest lifting
RO-Z2055

Central brake system
RO-Z2054

Laterally bed carrier
RO-Z2081

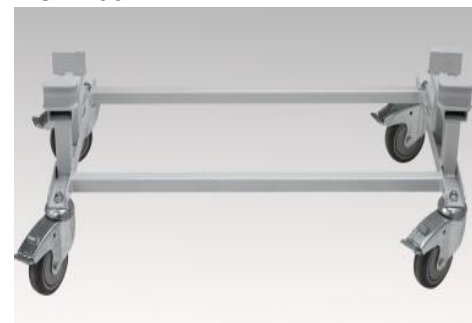

Seat depth - adjustable in 3cm steps from 45-57 cm
RO-Z2018

Grab handle on side rail
RO-Z2015

Cover for side rails and bars
RO-Z203PO



Shoulder bars / side rails back position
RO-Z2003R


Foot bars
RO-Z2003F




MATTRESSES

Wolfram Schweizer
 Dörrergasse 12
 90403 Nürnberg
 Telefon 0049-911-22 52 18
 Fax 0049-911-22 28 27
 info@wolframschweizer.de
 www.spezialbett.com

3- foam layer Trioptima

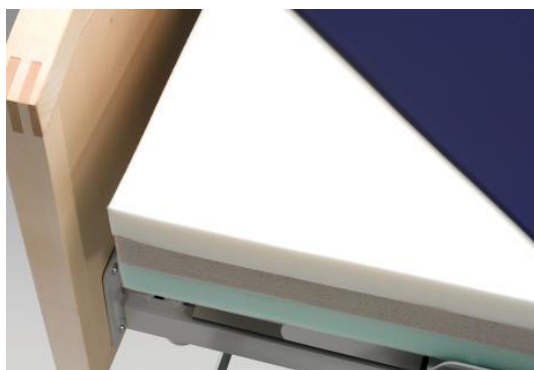

or 2-foam layer like illustration

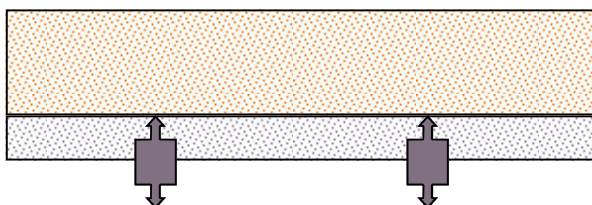

2-foam layer for following mattresses:
[Visco, Tempur und coldfoam](#)

Upper side: Visco, Tempur or coldfoam 8 cm oder 11 cm
 Lower side: carrier foam with an anchoring system



2- parted air dynamic mattress

Hico decubimat 380 for patient surface
 footrest in visco foam


The Hico features a large-cell, 2-chamber alternating pressure mattress for the pain-relieving positioning of patients (40 kg up to 160 kg weight) suffering from pressure ulcers



2- parted mattress with

3-D tissue and visco-foam cover for highest
 sleeping quality
 footrest in visco foam

3D-Tissue with oscillation - Oscillation as accepted
 therapy procedure

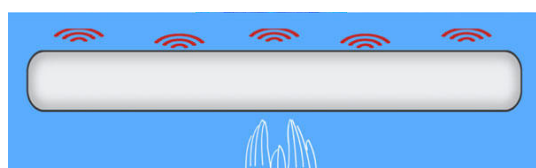

Oscillation therapy
 for the 3D-mattress


The oscillation could help you by different physical complaints:

- Supports relaxation by back pain
- Promotes the sleeping through every night
- Upset the lymphflux
- Reduces the water jam in the legs
- Tightens the skin, helps against cellulite
- Promotes the blood circulation
- Relieves ischia complaints
- Supports the liquid absorption of the spinal disc

The oscillation therapy is assisting the self healing for a better life

Only refrofitable by our technican or must be ordered with the bed

Wolfram Schweizer
 Dörrergasse 12
 90403 Nürnberg
 Telefon 0049-911-22 52 18
 Fax 0049-911-22 28 27
 info@wolframschweizer.de
 www.spezialbett.com

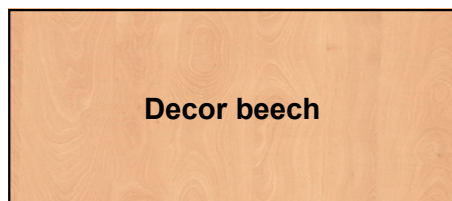

Decor beech

Divan Type 1

Or choosing other woods and colours at request or by sample




Decor maple

Divan Type II




Decor oak


Divan Type III

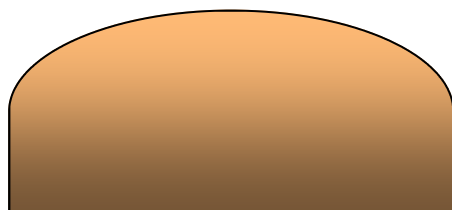

Rondo Type 1

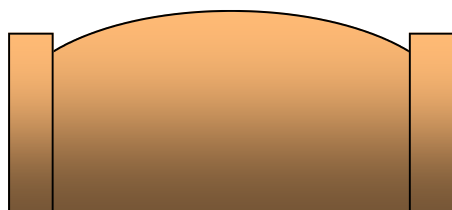

Rondo Type II

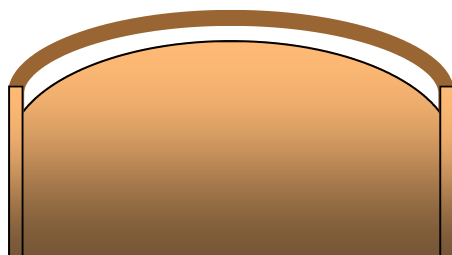



Rondo Type III



Pricelist 09/2018
Exw. Germany
Without packaging

**Compile your RotoFlex and mark it with a
and send it to info@wolframschweizer.de
Or with fax to 0049 911 222 827**



code-Nr.	text	Export EURO	Export EURO
RotoFlex	<u>indicate rotation direction</u> : clockwise  anticlockwise 	1 piece/order	10 pieces/order
RO-1003-H-125 Decor beach, oak or white	RotoFlex H , 5 section, 4 in 1, bed, chair, transfer- and standing aid, with white cover under the lathfloor, head-and foot-board with stainless steel reling, boards in beech or colour to RAL. Headboard removable. Twin-Castors 125mm, central brake with directional brake , lathfloor 90x210mm.	12.500,00 €	11.500,00 €
RO-1003-H-100 Decor beach, oak or white	RotoFlex H , 5 section, 4 in 1, bed, chair, transfer- and standing aid, with white cover under the lathfloor, head-and foot-board with stainless steel rail, boards in beech or colour to RAL. Headboard removable. Twin-Castors 100 mm, central brake , lathfloor 90x210mm.	12.335,00 €	11.350,00 €
RO-1003PSP Rondo Typ 1	RotoFlex P , 5 section, 5 section, 4 in 1, bed, chair, transfer- and standing aid, with white cover under the lathfloor but castors 75 mm, lathfloor 90x210mm. wheels with single break	12.335,00 €	11.350,00 €
RO-Z2017	Height adjustable footrest	800,00 €	740,00 €
RO-Z2014K	Pillow pocket, washable 90°	80,00 €	75,00 €
RO-Z2002-M	side-rail for right side middle position	187,00 €	157,00 €
RO-Z2003-M	side-rail for left side middle position	187,00 €	157,00 €
RO-Z2002-R	side-rail for right side back-position / shoulder bars	170,00 €	152,00 €
RO-Z2003-R	side-rail for left side back-position / shoulder bars	170,00 €	152,00 €
RO-Z2004 1/32S	lifting pole with handle standard	150,00 €	127,00 €
RO-Z2034	infusion pole with adapter	436,00 €	394,00 €
RO-Z2035	urin-bag holder	83,00 €	78,00 €
RO-	Foot bars / side rail	170,00€	170,00€
RO-Z2015	Get up handle on side rail	160,00€	150,00€
RO-Z2037	Patient surface in 100 cm width	270,00€	260,00€
RO-Z2040	Patient surface extension to 220 cm	250,00€	240,00€
Bedframes	By choosing another bedframe as Rondo I or Divan I Décor in beech, oak, maple and white have the similar price	+Additional charge	+ Additional charge
RO-Z2022 PI	wooden bedframe, beech, type RONDO PI		
RO-Z2022 PII	wooden bedframe, beech, type RONDO PII	300,00 €	200,00 €
RO-Z2022 PIII	wooden bedframe, beech, type RONDO PIII	1.100,00 €	1.000,00 €
RO-Z2024 PII	Homely wooden bedframe with cutout, Rondo II	1.500,00€	1.400,00€
RO-Z2023 PI	wooden bedframe, beech, type DIWAN PI		
RO-Z2023 PII	wooden bedframe, beech, type DIWAN PII	800,00 €	700,00 €
RO-Z2023 PIII	wooden bedframe, beech, type DIWAN PIII	800,00 €	700,00 €
RO-Z2025 P II	Homely wooden bedframe with cutout DIWAN II	1.400,00 €	1.300,00 €
RO-Z2054	Central brake system with 100 mm castors Central brake system with 125 mm castors	750,00 € 1.050,00 €	700,00 € 980,00€

Schweizer
Spezialbetten · Spezialsessel^{e.K.}

Wolfram Schweizer
Dörrergasse 12
90403 Nürnberg
Telefon 0049-911-22 52 18
Fax 0049-911-22 28 27
info@wolframschweizer.de
www.spezialbett.com

**My add up
Page 1**

Subsequent delivery costs are depending on place/country

Compile your RotoFlex and mark it with a

code-Nr.	text	EURO	EURO
Matresses	mattress with PU laminated incontinence-cover airper meable and anchorsystem, washable up to 90°C, serial height 11 cm	1 piece/order Ex works	10 pieces/order Ex works
RO-Z20S1-N	foam-mattress, normal strength, for bodyweight 50-90kg	527,00 €	500,00 €
RO-Z20S1-F	foam-mattress, normal strength, for bodyweight over 90kg	578,00 €	525,00 €
RO-Z20P1-N	foam-mattress,HD-foam airpermeable, normal strength, for bodyweight 50-90kg	640,00 €	610,00 €
RO-Z20P1-F	foam-mattress,HD-foam airpermeable, normal strength, for bodyweight over 90kg	670,00 €	640,00 €
RO-Z2TR1-N	foam-mattress TRIOPTIMA® 3-layer-foam with HD-foam + viscoelastic foam, body-weight 50-90 kg, height 11 cm Optional height 14 cm	986,00 € 1.250,00€	950,00 € 1.200,00€
RO-Z2TR1-F	foam-mattress TRIOPTIMA® 3-layer-foam with HD-foam + viscoelastic foam, for body-weight over 90 kg, height 11cm Optional height 14 cm	1.050,00 € 1.350,00€	1.000,00 € 1.250,00€
RO-Z20T1-N	mattress with Tempur-foam, bodyweight from 50 to 100 kg	1.510,00 €	1.400,00 €
RO-Z20T1-F	Mattress with Tempur-Foam, bodyweight over 100 Kg	1.800,00€	1.700,00€
RO-ZDWS-HI	2- parted air dynamic mattress Hico decubimat 380 for patient surface, footrest in visco foam, Subsequent delivery only by our technican or must be ordered with the bed	5.500,00€	5.200,00€
RO-Z20V1-N	mattress with viscoelastic-foam, 2-layer foam for bodyweight from 50 kg up to 100 kg, height 11 cm Optional height 14 cm	850,00 € 1.150,00€	800,00 € 1.100,00€
RO-Z20V1-F	mattress with viscoelastic-foam, 2-layer foam for bodyweight over 100 kg, height 11 cm Optional height 14cm	900,00 € 1.200,00€	850,00 € 1.150,00€
RO-Z0W3-D Our best RO-Z0W3-DOS	2- parted mattress with 3-D tissue and visco-foam cover for highest sleeping quality , footrest visco foam, height 14cm 3D-Tissue with oscillation - Oscillation as accepted therapy procedure + additional charge	2.000,00€ +1.400,00€	1.800,00€ +1.300,00€
Oscillation	Not upgradeable, must be ordered with the bed & mattress		
RO-Z2011-F	stretch-sheet frottee, set: large and small part	82,00 €	79,00 €
RO-Z2011-J	stretch-sheet Jersey, set: large and small part	84,00 €	81,00 €
RO-Z2012	mattress incontinence cover, large+small part	263,00€	243,00€
RO-Z2011-I	incontinence-stretch-sheet, beige, set: large and small part	96,00 €	83,00 €
RO-Z203PO	Cover for side rail bars, each side part	180,00€	170,00€
RO-Z2014K	holder-bag for head pillow	78,00 €	75,00 €
packaging	Ready mounted for use in a wooden box for sea + airfreight	300,00 €	250,00 €



Wolfram Schweizer
 Dörrergasse 12
 90403 Nürnberg
 Telefon 0049-911-22 52 18
 Fax 0049-911-22 28 27
 info@wolframschweizer.de
 www.spezialbett.com

My add up page 2

My add up Page 1

My add up complete

Please give us your dates and we send you a personalized offer for your bed.

Wolfram Schweizer
Dörrergasse 12
90403 Nürnberg
Telefon 0049-911-22 52 18
Fax 0049-911-22 28 27
info@wolframschweizer.de
www.spezialbett.com

Name _____

Address _____

Town _____

Country _____

Telephon & Email & Fax _____

Next airport or seaport _____

Your size and weight _____

Disease _____

Wishes & needs _____

When you lie in the bed
the bed should turn clockwise anticlockwise
 right left

Please print the pages 16, 17 and 18 and send your estimated demand per email scan to info@wolframschweizer.de or with Fax to 0049 911 222 827

Please note:

We do our best to personalize the bed for your needs but good things need time.

The delivery time is about **6 weeks EX works plus carrier** after clearing of all details and entrance of deposit and irrevocable letter of credit or complete payment in advance.

Guarantee is one year. In this case we send spare parts.

Delivery time **personalized**: 6 weeks EX works
Air freight ~ 10 days
Sea freight ~ 5 weeks

Delivery time **in stock**: 5-8 days
Air freight ~ 10 days
Sea freight ~ 5 weeks

How you get your RotoFlex bed




Wolfram Schweizer
Dörrersgasse 12
90403 Nürnberg
Telefon 0049-911-22 52 18
Fax 0049-911-22 28 27
info@wolframschweizer.de
www.spezialbett.com


A secure packaging for your RotoFlex bed, ready mounted for use. Only unpacking, plug in socket and ready!



We take care about the socket and the current voltage in your country with universal adapter

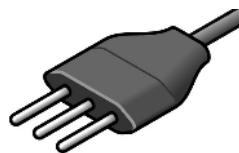

How to carry 190 Kg

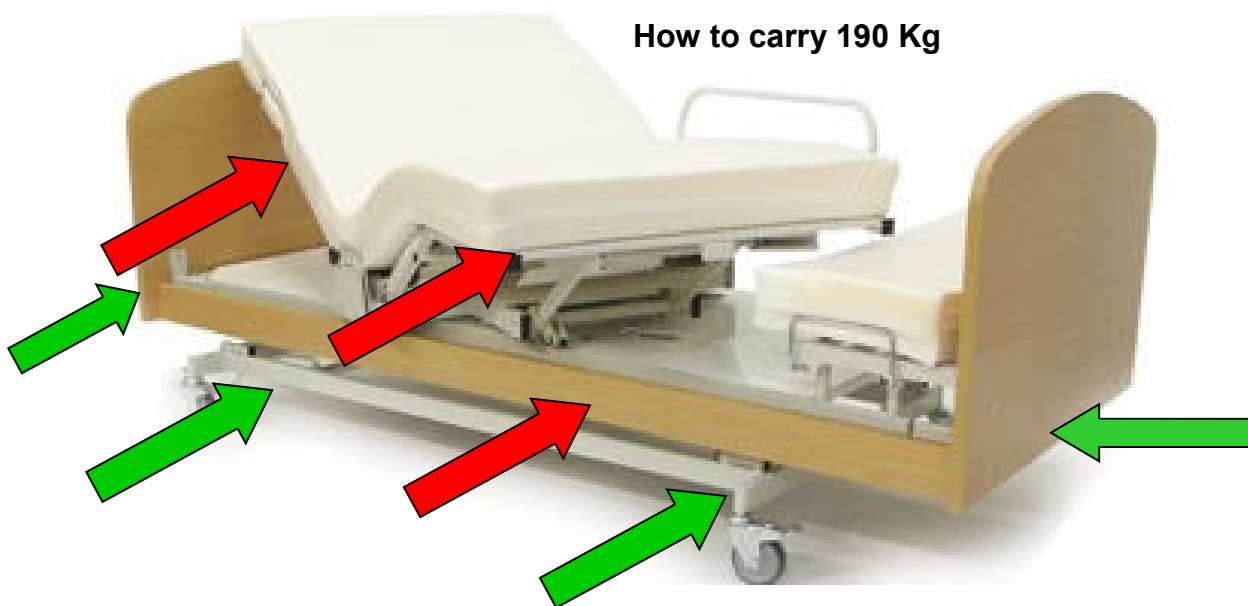

	Carry the RotoFlex here
	Don't carry it here